

Patterns

Turning data into discussion

RocheDiabetes Care Platform allows you to quickly and easily visualize your patients' self-monitored blood glucose data to discover patterns and draw attention to areas that might need intervention.





RDCP provides visual indicators for **up to 21 different glucose patterns** organized into five predefined pattern groups:

- Hypoglycemia
- Hyperglycemia
- Variability
- Use of System
- Treatment Adherence

With Patterns, you can create a collaborative conversation around your patients' self-monitoring. Acknowledge positive patterns as well as those outside desired thresholds, and discuss causal factors and potential changes for therapy optimization.




Glucose measurements that trigger any enabled patterns are highlighted in red or blue in the graph, and group indicators are shown next to each pattern group.

With help from RDCP and Patterns, you can make consultations more efficient and help foster patient learning and engagement.



How to set up Patterns

on the RocheDiabetes Care Platform

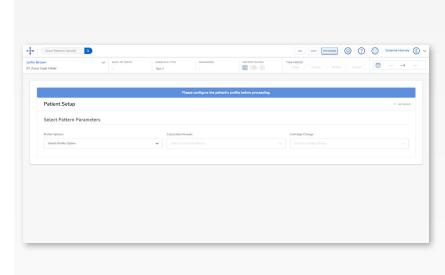


First, click on the **Patterns Icon** on the top menu navigation bar on the **Patient Dashboard.**

If this is the first time you are accessing **Patterns**, you will be directed to the **Patient Setup screen**.

- 2 Configure the **Pattern Parameters** for your patient.
- 3 Review the Minimum Tests Required to view a pattern and the Glucose Target Ranges. These are initially predefined but can be customized for each patient.

- The Configure Patterns section will allow you to select up to 21 different glucose patterns organized into five groups: Hypoglycemia, Hyperglycemia, Variability, Use of System, and Treatment Adherence.
- 5 In the **Configure Patterns** section, you can also customize exactly when RDCP will highlight a pattern.
- 6 After completing your Patient Setup, click Save Configuration and confirm.
- 7 Once completed and saved, the Patterns Icon will now open the Patterns Dashboard for your patient.

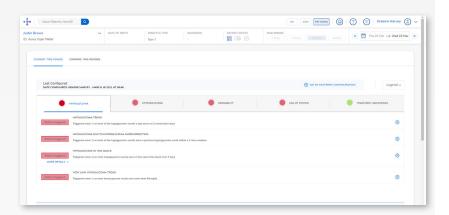

Remember, a minimum number of tests are required for results to be calculated, and data upload from a patient's device is required for results to display on the **Patterns Dashboard.**

For additional information log into your HCP Account and click the ② on the main menu bar.

Roche Diabetes Care is aware the platform may need to undergo a Data Privacy Impact Assessment. Please contact your Roche Sales Representative if you would like assistance during this process.

ACCU-CHEK, ROCHE DIABETES CARE, and ROCHEDIABETES CARE PLATFORM are trademarks of Roche. All other product names and trademarks are the property of their respective owners. © 2021 Roche. 348-56685A-0321



Want more information?