



# All solutions in one place



1. Install the mySugr app



2. Pair your meter



3. Share data with your doctor

**We want to support you in simplifying your diabetes management.**

With just one single account, you can now access your health data, share it with your healthcare professional and maintain transparency over your health journey.


**Download the mySugr® app**

It's easy. Scan the code with your phone to download, then use your email to create an account.



A

# Install the mySugr app and create your account

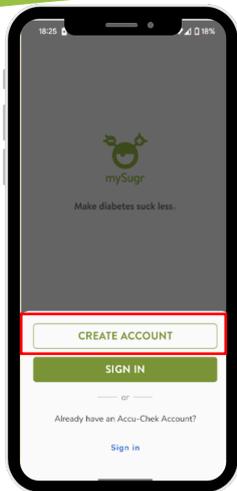

1


After you downloaded the mySugr app from the App Store, tap on "Create Account" to create your mySugr account.

Note: If you already have a mySugr account select "Sign In". If you are already logged into mySugr, then proceed to Step 2.




2

After logging into the mySugr app, select the "More" tab in the bottom right corner.

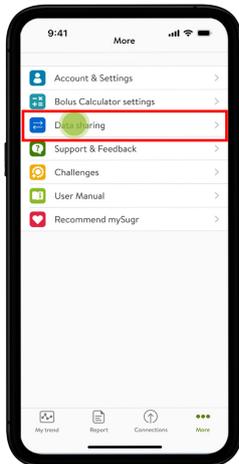


3

Then you will select "Data Sharing" to begin your Accu-Chek account creation process.

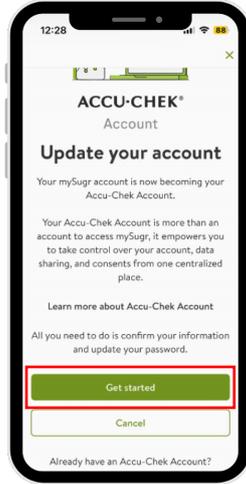


4

Select "Get Started" to create your Accu-Chek Account. This new Accu-Chek Account will allow you to create a sharing code with your healthcare provider.

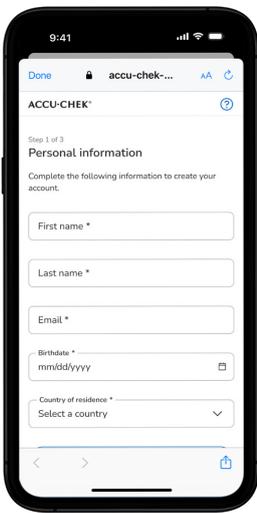


5

Begin by entering your personal information so that your healthcare professional can identify your patient profile.

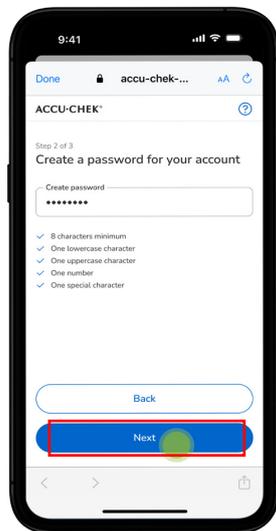


6

Create a password following the password requirements listed on screen. After you create your password, select "Next".

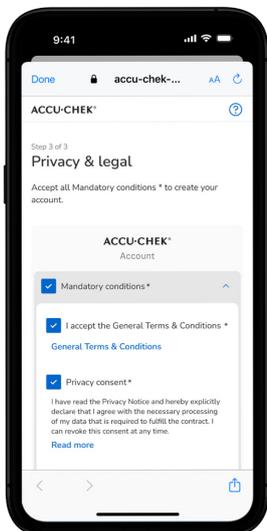


7

Review privacy and legal consents.

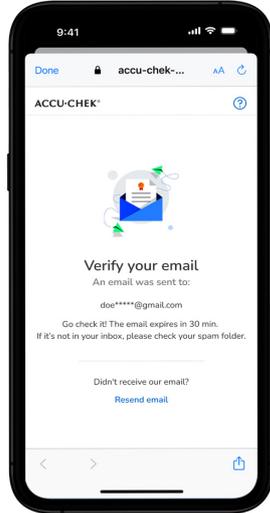


8

A verification link is then sent to your email.

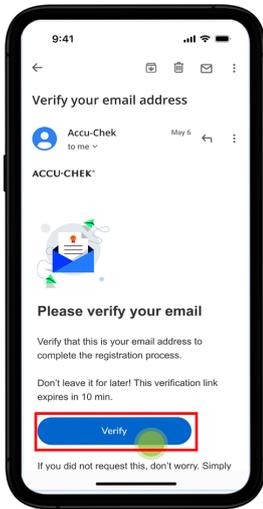


9

Verify your email via the shared link. Return to the mySugr app to complete setting up your account.

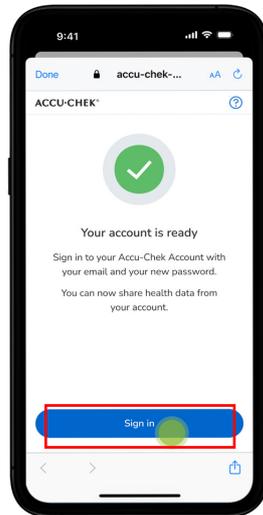

10

Your account set up is now complete. You may now select "Sign In".

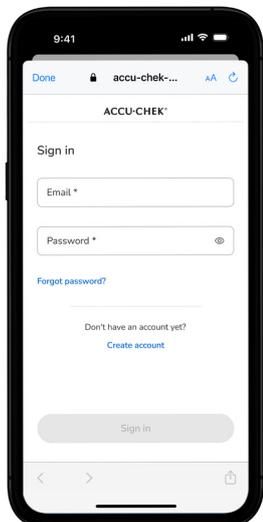

11

Congratulations! You can now login to access your new Accu-Chek account.

Note: This new login will replace your previous mySugr account login. You will now login to mySugr using your Accu-Chek account credentials.


B

# Pair your Accu-Chek blood glucose meter




1

Make sure both your phone's and your Accu-Chek meter's bluetooth is turned on. To connect your blood glucose meter, tap the **"Connections"** tab at the bottom right of the screen.

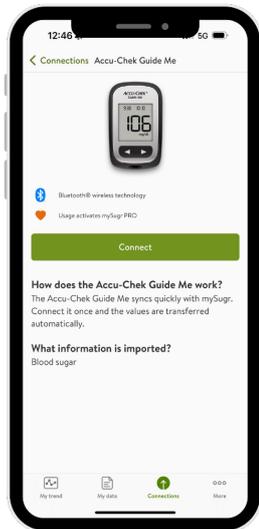



2

Choose the blood glucose meter you want to connect and tap **"Connect"**.

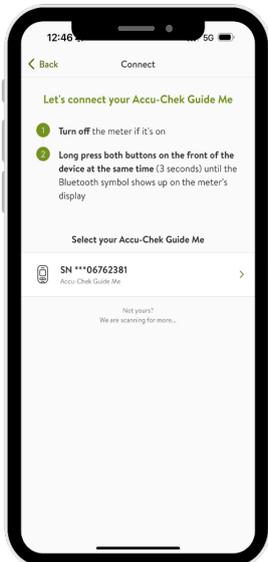

3

Please follow the steps provided on screen to put your device into pairing mode.

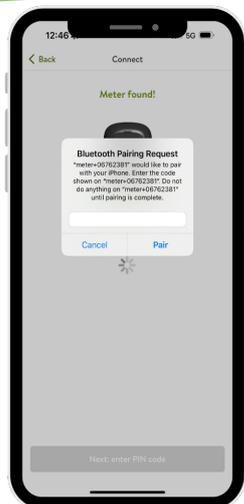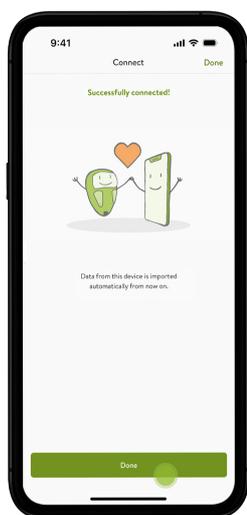

4

**Next**, you will receive a bluetooth pairing request which requires you to enter the PIN on the back of your meter.

Once you entered the PIN, tap **“Pair”**.

5

**Congratulations!**

You have successfully connected your meter to the mySugr app.

Once you pair your Accu-Chek blood glucose meter with mySugr, the data from your meter will automatically be imported into the app.






# Share your data with your healthcare professional\*




1

To connect with the Accu-Chek Care platform, please head to the home screen in the mySugr app and tap **“More”**.

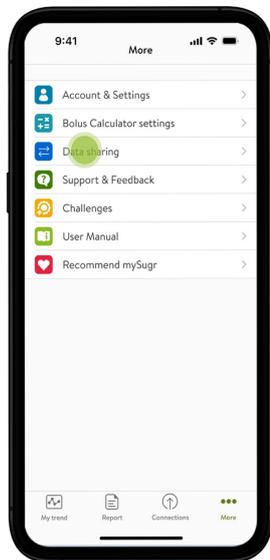


2

Tap **“Data sharing”** to initiate the sharing of your diabetes information.

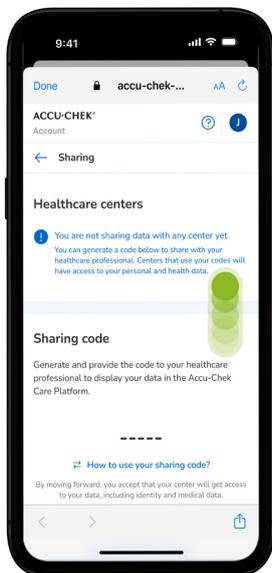

3

On the sharing tab via your Accu-Chek account, swipe down to go to the sharing code section.

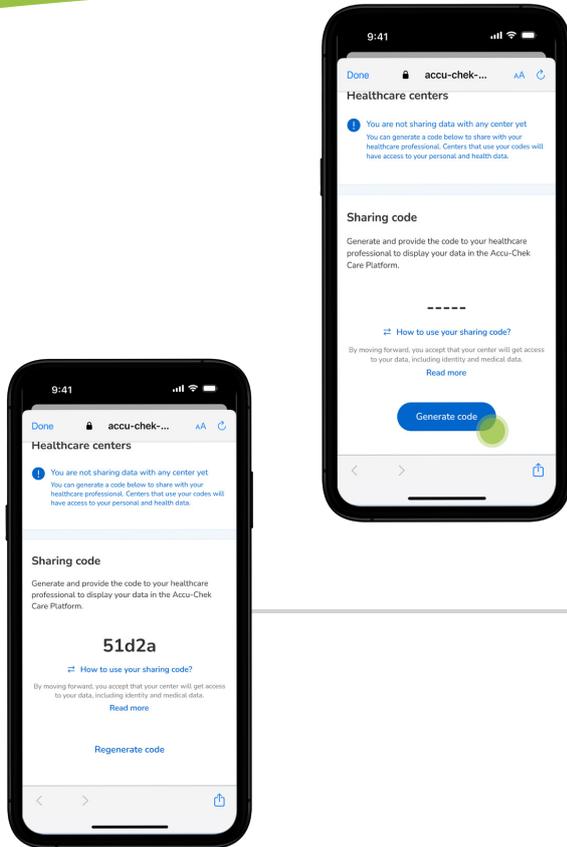

4

Tap on **“Generate Code”**

5

**Once your code has been generated, share the code with your healthcare professional to give them access to your diabetes data.**

\*Please make sure that your healthcare provider is using Accu-Chek Care platform to ensure a safe connection. In case your healthcare professional does not have the Accu-Chek Care platform, you can still share your PDF report downloaded from your mySugr app. **Please note:** You must be at least 18 years old to register for the Accu-Chek Account.